
# LiNA OperåScope™


## Scissors

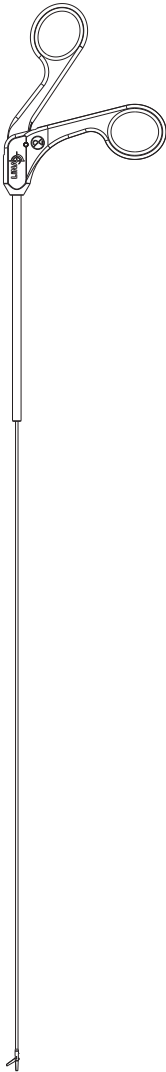RxOnly

Refer to the Symbol Definition Chart FV0223, the integral part of Instructions for Use, for the meanings of the symbols.
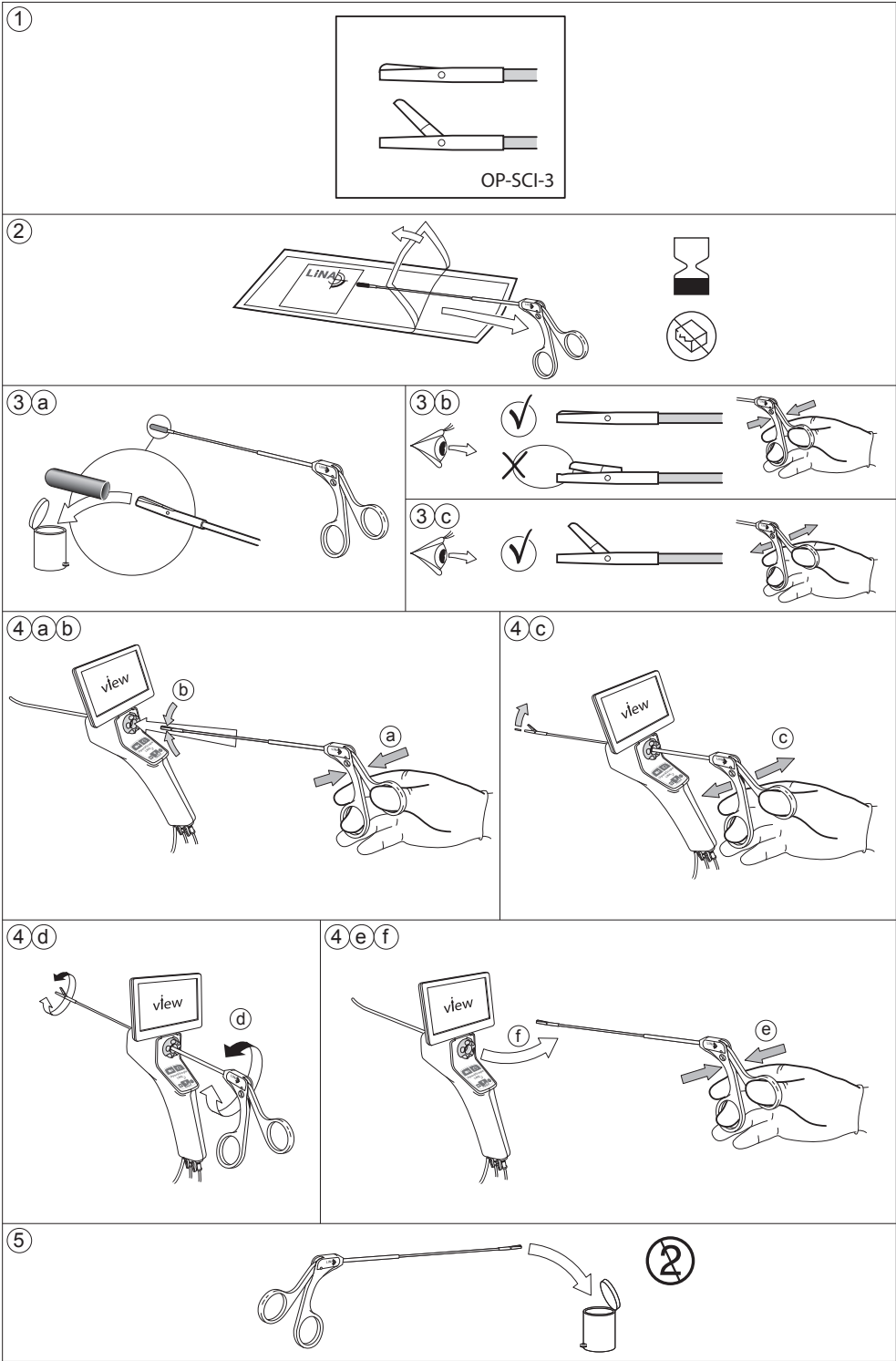
## LiNA OperâScope™ Scissors

REF OP-SCI-3

Refer to the Symbol Definition Chart FV0223, the integral part of Instructions for Use, for the meanings of the symbols.

en

### Product description:

The Scissors are delivered as a sterile, single-use device. The Scissors are designed to be used with the LiNA OperâScope™ hysteroscope. The device consists of handles, a shaft and the blade system (scissors) at the distal end. The Scissors are compatible with hysteroscopes with a straight working channel of min Ø1.87mm (5.6Fr) diameter and a max length of 310mm.

### Intended use:

The LiNA OperâScope™ Scissors (Scissors) are intended for use in hysteroscopy procedures to cut soft tissue such as endometrial polyps, intrauterine adhesions or septa.

### Indication for use:

Use in hysteroscopy procedures to cut soft tissue such as polyps or the endometrium in the uterine cavity.

### Patient population:

Adolescent and adult females suited for hysteroscopy.

### Contraindications:

This product is not intended for use when hysteroscopy is contraindicated. Contraindications for hysteroscopy include:

- Acute Pelvic Inflammatory Disease (PID)

Hysteroscopy may be contraindicated by the following conditions, depending on their severity or extent:

- Inability to distend the uterus
- Cervical Stenosis
- Cervical/Vaginal infection
- Uterine bleeding or menses
- Known Pregnancy
- Invasive carcinoma of the cervix
- Recent uterine perforation
- Medical contraindication or intolerance to anesthesia

### Potential complications:

Potential complications of Continuous Flow Hysteroscopy include: uterine perforation/false passage (resulting in possible injury to bowel, bladder, major blood vessels and ureter), cervical laceration, hemorrhage, vasovagal

## LiNA OperâScope™ Scissors

REF OP-SCI-3

Refer to the Symbol Definition Chart FV0223, the integral part of Instructions for Use, for the meanings of the symbols.

en

### Precautions:

- Use of the LiNA OperâScope™ Scissors requires adequate training and experience in performing hysteroscopy.
- Always have a backup device readily available for immediate use.
- If any malfunction should occur during use, stop the procedure immediately, and slowly withdraw the LiNA OperâScope™ Scissors and replace with a new device.
- Manipulate with the forces required to cut the tissue, and do not apply excessive force. Over-exertion could cause the Scissors to fail. Do not pinch hard tissues, foreign bodies, or other devices.
- After use, the device is to be considered contaminated medical waste and may pose a potential infection risk. Dispose of the device and packaging in accordance with applicable laws, administrative, and/or local government policy and hospital procedures, including those regarding biohazards, microbiological hazards and infectious substances.
- The LiNA OperâScope™ Scissors are intended only as an adjunct in assessing patient condition. It must be used in conjunction with clinical signs and symptoms.

### Instruction for use:

- Inspect the label to ensure that the correct device is chosen (**Figure 1**).
- Using sterile technique, remove the device from the sterilized blister (**Figure 2**).
- Remove the protection cap (**Figure 3a**).
- Inspect the Scissor blades for any obvious damage.
- Check the open/close function of the Scissors by actuating the handle. Opening and closing the blade system is done by pushing or pulling the Handles from/ towards each other (**Figure 3b & 3c**).
- Close the Scissors (**Figure 4a**) before inserting the device into the LiNA OperâScope™ working channel (**Figure 4b**).

- Slowly insert the Scissors into the LiNA OperâScope™ working channel until the blades are visualized on the LiNA OperâScope™ live camera image. Some resistance will be felt when then blades system passes through LiNA OperâScope™'s pre-curved tip. Take care not to bend the Scissors shaft.
- Maneuver the Scissors towards the targeted site. Open the Scissors by activating the handles (**Figure 4c**). Advance the opened Scissors over to the targeted site, rotate the Scissors, if needed, (**Figure 4d**) and close the Scissors. Use only enough force to cut the tissue. Over-exertion could cause the Scissors to fail.
- Continue to apply gentle force on the handles (**Figure 4e**) as the Scissors are extracted from the working channel (**Figure 4f**).
- Steps 4-9 may be repeated if multiple cuts are needed within a single procedure.

### Reporting:

Any serious incident that has occurred in relation to the device should be reported to the LiNA Medical AG and the competent regulatory authority of the country in which the user and/or patient is established.

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